



180 PCB Controller

T565


Programming Instructions QUICK GUIDE
PCB Software Version 0.812 or higher

Last Update 22.03.2016


Accessing PROGRAM mode P1

Position the lift just below the top charge point.


Press and hold
4 seconds

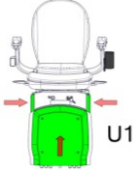

Press **UP** once

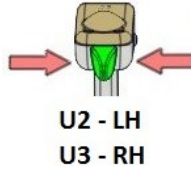


Press **UP** once more

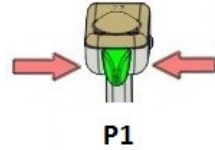

Level Carriage




Press **UP** safety edge **ONLY** once to store level reference

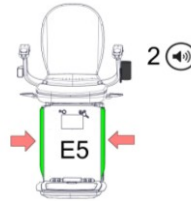


PROGRAMMING the rail P1-P2


Press **UP** to Top charge point

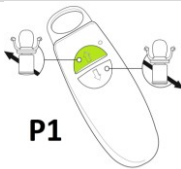


Set metal striker

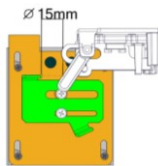


Press **UP** to Top Final Limit

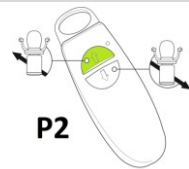


Travel back **DOWN** to Top charge point

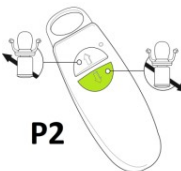


Press **UP/DOWN** to select speeds

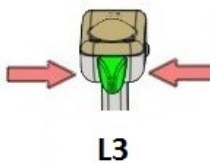

Press **DOWN** to start program

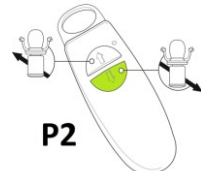


Info Approaching Vertical Bends MAX L3 speed

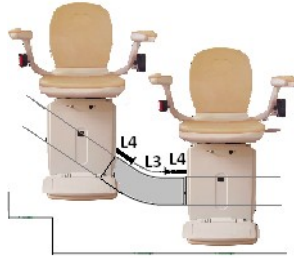


BOTTOM stop P3


Press **DOWN** to Bottom CP or HINGE

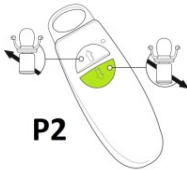


Using a charge point set metal striker

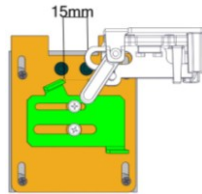

Press **DOWN** to Bottom Final Limit

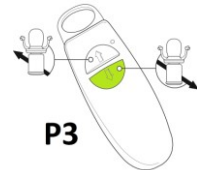


Using a **HINGE** activate limit switch by hand

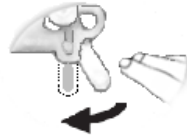

Resetting Self-Levelling safety mode J3

Press **UP** to Top charge point

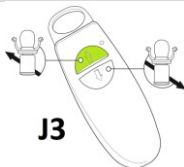

At the top charge point



Pairing the HINGE BOX P9 - if required



Away from the charge point

Swivel the seat

Press **DOWN SIDE only** and HOLD

Press **UP** once

In the Hinge Control Box

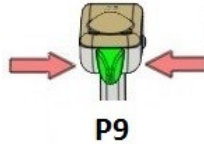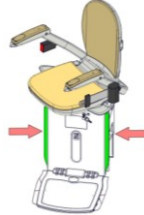

PRESS
●
BLACK BUTTON
5 SECONDS





Setting HINGE ACTIVATION POINT L8 - if required

Place the Stairlift on the selected Hinge activation point

Swivel the seat

Press **DOWN SIDE only** and HOLD




Press **UP** once

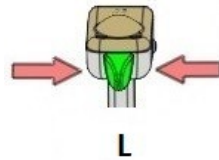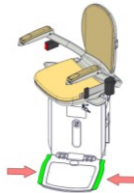

Select **L8**

Seat back

Press **DOWN** to confirm

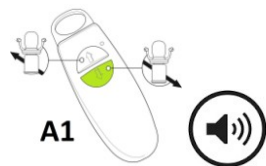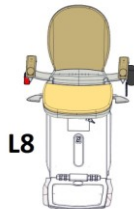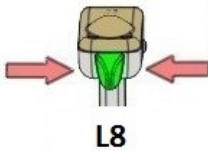


Changing to a different speed after programming

Position the Stairlift on the section that requires a speed change.


Swivel the seat




Press **DOWN SIDE** only and HOLD

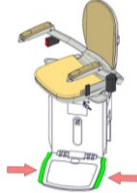


Press **UP** once

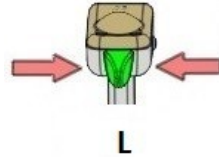


Select new **L** speed

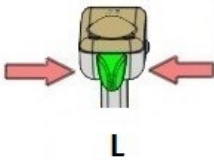

Swivel seat back to travel position

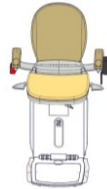

Press **DOWN** to confirm

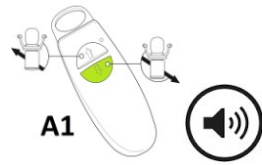


Changing the level reference after programming

Level the lift manually


Swivel the seat




Press **DOWN SIDE** only and HOLD

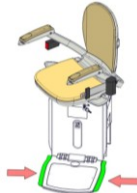


Press **UP** once

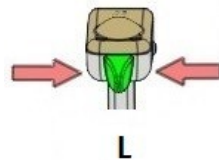

Select new **L9**

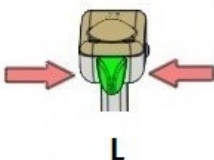

Swivel seat back to travel position

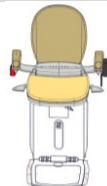

Press **DOWN** to confirm



Changing the *VOLUME* of Off-Charge beeper

Off charge, swivel the seat

Press **UP** or **DOWN** to increase or decrease




Accepting another remote control

On a charge point


Swivel the seat

Press **DOWN safety** and **HOLD**

Press **DOWN** to confirm

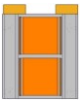

C1/C4

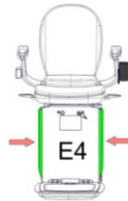

Clearing all remotes stored


On a charge point

Swivel the seat

Press **UP and DOWN carriage** and **footrest edges** and **HOLD**



C1/C4

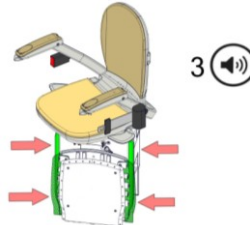

Reading Battery voltages, Software level and Serial number

Off charge, swivel
the seat



Press **UNDERPLATE**
safety edge



Sequence displayed

- 1st Left battery
- 2nd Right battery
- 3rd U1 software
- 4th U2 software
- 5th U8 software
- 6th PCB serial number